

# RADIESSE®

## Skin Vitality, Re-Awakened Beyond the Face

Re-Awaken skin from within with RADIESSE®,  
the first and only U.S.-approved biostimulator  
for décolleté<sup>1\*</sup>


Please see **Important Safety Information** for  
RADIESSE and RADIESSE (+) throughout, and visit  
[www.RADIESSE.com](http://www.RADIESSE.com) for RADIESSE/RADIESSE (+)  
**Instructions For Use.**

Actual patient. Individual results may vary.

\*RADIESSE is now FDA-approved for the correction of décolleté wrinkles in  
patients aged 22 years and older when diluted with sterile saline.<sup>1</sup>

# Go Beyond Surface Solutions

One of the first areas to show signs of aging is the décolleté.<sup>2</sup> RADIESSE® replenishes skin's natural reserves of collagen, elastin, and proteoglycans.<sup>3-6</sup>

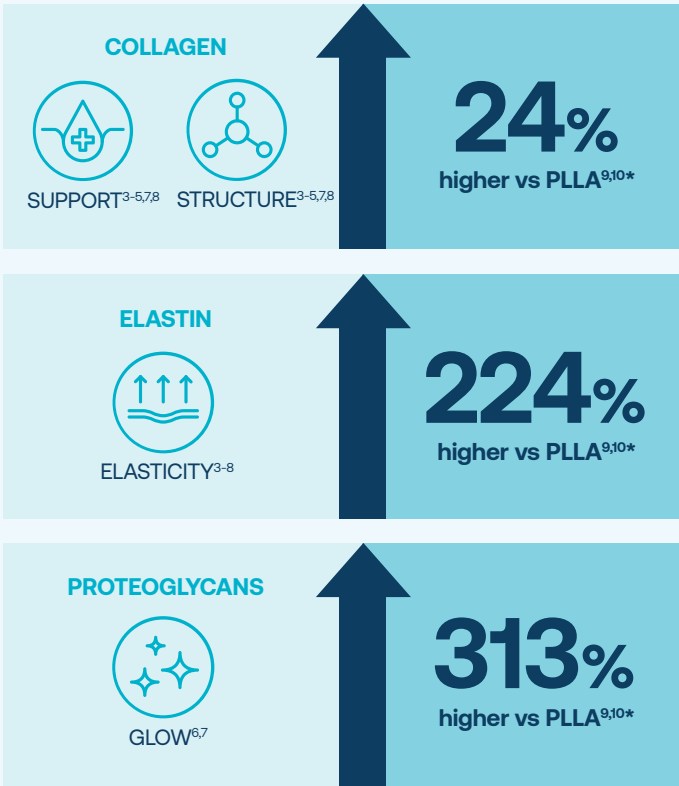

PLLA=poly-L-lactic acid.

\*Data based on laboratory studies of skin tissue comparing RADIESSE® and Sculptra® after 6 months.<sup>9</sup>

## What are RADIESSE® and RADIESSE® (+)?

RADIESSE® and RADIESSE® (+) are dermal fillers that are used for smoothing moderate to severe facial wrinkles and folds, such as nasolabial folds (the creases that extend from the corner of your nose to the corner of your mouth).

**RADIESSE®**

# Rebuild Skin From Within

RADIESSE enhances skin quality by working with your body to strengthen skin structure and improve skin health.<sup>9</sup> Results are long-lasting\* and look natural by stimulating your body's own production of collagen, elastin, and proteoglycans—it's your skin, re-awakened from within.<sup>3-8</sup>

\*"Long-lasting" indicates results that may last 84 weeks.<sup>10</sup>

## The Reversal Is Real: See the Visible Difference in the Décolleté

This patient achieved visibly striking results after 6 months with a personalized treatment plan. **Your provider will help determine the right approach for you.**



Patient aged 69 years, treated with RADIESSE diluted 1:2 with saline at initial treatment. Patient was then treated with RADIESSE diluted 1:2 with saline 6 weeks later at a follow-up appointment. Image was not retouched. Individual results may vary.

## What are RADIESSE® and RADIESSE® (+)? (cont'd)

RADIESSE® is also used for correcting volume loss in the back of the hands and, when diluted with saline to smooth away wrinkles such as those in the décolleté in patients 22 years of age and older. RADIESSE® (+) is also used for improving moderate to severe loss of jawline contour in adults over 21 years old.

Please see **Important Safety Information** for RADIESSE and RADIESSE (+) throughout, and visit [www.RADIESSE.com](http://www.RADIESSE.com) for RADIESSE/ RADIESSE (+) **Instructions For Use**.

# Rediscover Visibly Healthy Skin on Your Décolleté

Over 80% of people reported...



Visible improvement may last 1.5+ years after treatment<sup>10</sup>



Satisfaction with their appearance after treatment compared to before<sup>10</sup>



Significant improvement of fine lines and wrinkles<sup>10</sup>

## **RADIESSE® and RADIESSE® (+) IMPORTANT SAFETY INFORMATION**

### **Who should not use RADIESSE® or RADIESSE® (+)?**

You should not use RADIESSE® or RADIESSE® (+) if you have an allergy to any component of the product, if you have a history of severe allergies, if you have a bleeding disorder. You should not use RADIESSE® (+) if you have an allergy to lidocaine or medicines like it.

**RADIESSE®**



Actual patient. Individual results may vary.

Please see **Important Safety Information** for RADIESSE and RADIESSE (+) throughout, and visit [www.RADIESSE.com](http://www.RADIESSE.com) for RADIESSE/ RADIESSE (+) **Instructions For Use**.

## **What are RADIESSE® and RADIESSE® (+)?**

RADIESSE® and RADIESSE® (+) are dermal fillers that are used for smoothing moderate to severe facial wrinkles and folds, such as nasolabial folds (the creases that extend from the corner of your nose to the corner of your mouth).

RADIESSE® is also used for correcting volume loss in the back of the hands and, when diluted with saline to smooth away wrinkles such as those in the décolleté in patients 22 years of age and older. RADIESSE® (+) is also used for improving moderate to severe loss of jawline contour in adults over 21 years old.

## **RADIESSE® and RADIESSE® (+) IMPORTANT SAFETY INFORMATION**

### **Who should not use RADIESSE® or RADIESSE® (+)?**

You should not use RADIESSE® or RADIESSE® (+) if you have an allergy to any component of the product, if you have a history of severe allergies, if you have a bleeding disorder. You should not use RADIESSE® (+) if you have an allergy to lidocaine or medicines like it.

### **What is the most important information I should know about RADIESSE® and RADIESSE® (+)?**

One of the risks with using these products is unintentional injection into a blood vessel. The chances of this happening are very small, but if it does happen, the complications can be serious and may be permanent. These complications, which have been reported for facial injections, can include vision abnormalities, blindness, stroke, temporary scabs, or permanent scarring of the skin. If you have changes in your vision, signs of a stroke (including sudden difficulty speaking, numbness or weakness in your face, arms, or legs, difficulty walking, face drooping, severe headache, dizziness, or confusion), white appearance of the skin, or unusual pain during or shortly after treatment, you should notify your health care practitioner immediately.

As with all procedures that involve an injection through the skin, there is a risk of infection.

Do not use RADIESSE® or RADIESSE® (+) if you have a skin infection until it has healed.

It is not known if RADIESSE® or RADIESSE® (+) is safe or effective in the lips, or in the area around the eyes, or if you are pregnant or breastfeeding.

It is not known if RADIESSE® or RADIESSE® (+) is safe or effective beyond 3 years in the face, 1 year in the hand, and 84 weeks in the décolleté.

Injection of RADIESSE® (+) into the jawline may temporarily alter jaw function.

The safety of RADIESSE® in the décolleté in patients under 22 years of age has not been established.

The microspheres in RADIESSE® and RADIESSE® (+) can be seen in X-rays and CT scans. It is very important that you tell your health care provider that you have had RADIESSE® or RADIESSE® (+) dermal filler. A clinical study found no evidence

of RADIESSE® injected in the décolleté being visible on mammograms or breast ultrasounds, but you should still inform your doctor and radiologist that you have received RADIESSE® in the treated area so they can interpret your imaging appropriately.

If you have a history of herpes, you may experience a herpes breakout after receiving RADIESSE® or RADIESSE® (+).

Injection in the back of the hand may result in temporary difficulty performing activities. RADIESSE® may cause nodules, bumps or lumps in the back of the hand and can last up to 1 year.

You should minimize strenuous activity and avoid extensive sun or heat exposure for about 24 hours after treatment and until any swelling or redness has resolved.

### **What should I tell my doctor before using RADIESSE® or RADIESSE® (+)?**

Tell your health care provider if you are taking blood thinners or medicines that can interfere with the clotting of blood, such as aspirin or warfarin. These medicines might make it more likely that you will experience bruising or bleeding at the injection site. Tell your health care provider if you have any diseases, injuries or disabilities of the hand, if you have a history forming large, raised scars or if you have had any other skin treatments such as skin peels.

### **What are the most common adverse events with RADIESSE® or RADIESSE® (+)?**

The most common adverse events seen in clinical studies of RADIESSE® used in the hands include bruising, redness, swelling, pain, itching, nodules or bumps/lumps, difficulty performing activities, loss of sensation and other local side effects. The most common adverse events seen in clinical studies of RADIESSE® or RADIESSE® (+) used in the face include bruising, redness, swelling, pain, itching, difficulty chewing, and other local side effects.

Delayed-onset inflammation near the site of injection is one of the known adverse events associated with dermal fillers. Cases of delayed-onset inflammation have been reported to occur at the treatment site following viral or bacterial illnesses or infections, vaccinations, or dental procedures. Typically, the reported inflammation was responsive to treatment or resolved on its own.

These are not all of the possible side effects with RADIESSE® or RADIESSE® (+). Merz collects information about adverse events seen with RADIESSE® and RADIESSE® (+) outside of clinical studies. These events are included in the RADIESSE® and RADIESSE® (+) Patient Information Guide based on an assessment of seriousness and potential causal relationship to RADIESSE® or RADIESSE® (+). Please see the Patient Information Guide available at [www.radiesse.com](http://www.radiesse.com) for list of these events. Tell your health care provider about any side effects that bother you or do not go away.

Important: For full safety information, please visit [www.Radiesse.com](http://www.Radiesse.com) or call MyMerz Solutions at 1-844-469-6379.

RADIESSE® and RADIESSE® (+) are available by prescription only.

# RADIESSE®

## It's Time to Invest in Your Décolleté



### Regenerate from within

Activate skin's natural pathway to self-renewal for lasting results that are authentically yours<sup>8,10</sup>



### Reverse visible signs of aging

RADIESSE® helps renew skin quality and restore your natural contour<sup>4,11</sup>



### Maintain your look longer

Décolleté results may last 1.5+ years<sup>10</sup>

**Ready to firm, tighten, and reverse visible signs of aging?**

**Ask your injector about RADIESSE® to revitalize skin quality in the décolleté**

**References:** **1.** Radiesse [Instructions for Use]. Franksville, WI: Merz North America, Inc; 2026. **2.** Dénes Z. *Z Alternsforsch.* 1985;40(3):159-164. **3.** Yutskovskaya Y, et al. *J Drugs Dermatol.* 2014;13(9):1047-1052. **4.** Yutskovskaya YA, et al. *J Drugs Dermatol.* 2017;16(1):68-74. **5.** Yutskovskaya YA, et al. *J Drugs Dermatol.* 2020;19(4):405-411. **6.** González N, et al. *Dermatol Surg.* 2019;45(4):547-551. **7.** Nowag B, et al. CaHA microspheres: contact with fibroblasts and amount of spheres are key factors for collagen stimulation. Presented at: AMWC 2020; Monte Carlo, Monaco; April 2-4, 2020. **8.** Aguilera SB, et al. *Aesthet Surg J.* 2023;43(10):1063-1090. **9.** Casabona G, et al. Extracellular matrix response to diluted calcium hydroxylapatite microspheres and poly-L-lactic acid microparticles. Presented at: IMCAS 2025; Paris, France; Jan 30-Feb 1, 2025. **10.** Data on file. Merz North America, Inc. 2025. **11.** Moradi A, et al. *J Drugs Dermatol.* 2021;20(11):1231-1238.

Please see **Important Safety Information** for RADIESSE and RADIESSE (+) throughout, and visit [www.RADIESSE.com](http://www.RADIESSE.com) for RADIESSE/ RADIESSE (+) **Instructions For Use.**

**MERZ AESTHETICS®**

©2026 Merz North America, Inc. All rights reserved. MERZ AESTHETICS is a registered trademark of Merz Pharma GmbH & Co. KGaA in the U.S. RADIESSE is a registered trademark, and the R Logo is a pending trademark, of Merz North America, Inc. in the U.S. All other marks are the property of their respective owners. April 2026 US-RAD-2500161

Merz North America Headquarters  
6501 Six Forks Road, Raleigh, NC 27615